

CLEANSPACE HALO DOFFING ON STANDBY MODE



1. Remove harness over your head in a forward motion.



2. With your left hand, locate the mask clip and prepare to release.



3. To locate the Power Button – with your right hand, use your thumb to feel the battery charge point underneath – the raised Power Button will be directly above on the key pad.



4. When ready to doff, hold your breath and press the Power Button once ('beep') to switch unit to Standby Mode.



5. With the left hand, unclip and release the mask. Hold the mask with your left hand (do not let it fall to your chest).



6. Lean slightly forward, with your right hand remove the Power Unit from the back of the neck. Place the device down and out in front of you. Never remove the mask over your head.