
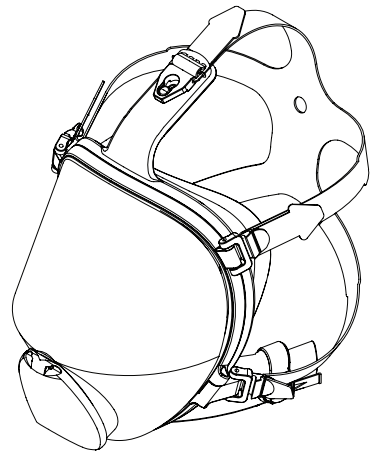
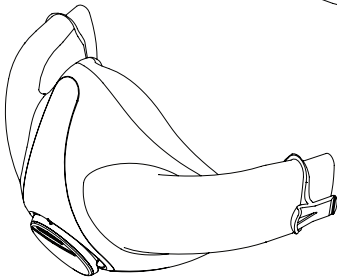
CleanSpace®

R E S P I R A T O R S

QUICK START GUIDE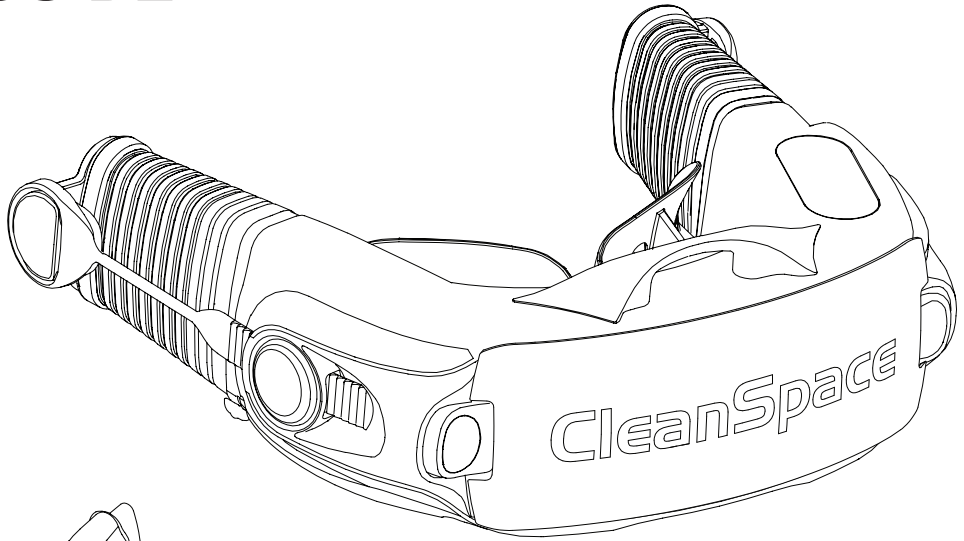

CleanSpace HALO

CleanSpace PRO

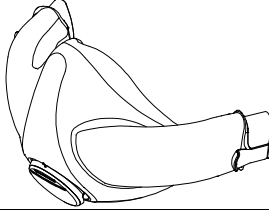
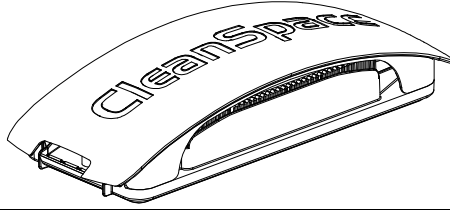
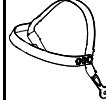
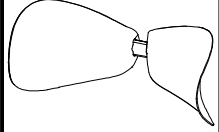
CleanSpace ULTRA

CleanSpace Technology Pty Ltd
ABN 24 146 453 554
T: +61 2 8436 4000

Unit 5, 39 Herbert Street
St. Leonards NSW 2065 Australia

www.cleanspacetechnology.com
sales@cleanspacetechnology.com

CleanSpace Respirator Approved Configurations

CleanSpace [®] RESPIRATORS																														
		CS3001	CST1002	CST1012	CS3003	CS3004	CS3005	CS3006	CS3007	CST1014	CST1015	CST1016	CST1017	CST1018	CS3002	CS3025	CST1004	CST1005	CST1006	CST1007	CST1008	CST1027	CST1028	CS3008	CST1021	CS3009	CS3010	CST1019	CST1020	PAF-1101
CleanSpace HALO	Outside USA	X			X	X	X	X	X					X	X									X		X	X			X
CleanSpace PRO	Europe		X						X	X	X	X	X			X	X	X		X	X	X		X			X	X	X	
	AUS / NZ		X						X	X	X	X	X			X	X		X					X			X	X	X	
CleanSpace ULTRA	Europe			X					X	X	X	X	X			X	X	X		X	X	X		X			X	X	X	
	AUS / NZ			X					X	X	X	X	X			X	X		X					X			X	X	X	

NOTE: USA users must consult the NIOSH Respirator Approval Label provided with each CleanSpace Respirator for the NIOSH-approved configurations

CleanSpace Respirator Approval Information

CleanSpace HALO		Notified Body: BSI (2797): Say Building, John M. Keynesplein, 1066 EP Amsterdam, Netherlands Standard: EN12942:1998/A2:2008 Classification: TM3 DOC: cleanspacetechnology.com/eu2016-425/ Regulation: (EU) 2016/425 Category: III Modules: B and D
		Notified Body: BSI (0086): Kitemark Court, Davy Avenue, Knowlhill, Milton Keynes, MK5 8PP Standard: EN12942:1998/A2:2008 Classification: TM3 DOC: cleanspacetechnology.com/eu2016-425/ Regulation: (EU) 2016/425 as brought into UK Law and amended Category: III Modules: B and D
CleanSpace PRO CleanSpace ULTRA		Notified Body: Apave Exploitation France SAS (0082): 6 Rue du Général Audran - 92412 COURBEVOIE Cedex - France Standard: EN12942:1998/A2:2008 Apave protocol 24.0145 dated 09/02/2024 Classification: TM3 DOC: cleanspacetechnology.com/eu2016-425/ Regulation: (EU) 2016/425 Category: III Modules: B and D
		Notified Body: PARC (8509): Unit 4, Alverdiscott Road Industrial Estate, Bideford, Devon, EX39 4LQ Standard: EN12942:1998/A2:2008 Apave protocol 24.0145 dated 09/02/2024 Classification: TM3 DOC: cleanspacetechnology.com/eu2016-425/ Regulation: (EU) 2016/425 as brought into UK Law and amended Category: III Modules: B and D

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CleanSpace Respirator Quick Start Guide

Please read the entire CleanSpace Respirator Quick Start Guide before using a CleanSpace Respirator. To view additional instructions and product information, go to cleanspacetechnology.com/product-resources/. Before use, refer to the relevant national standards in each local region and review methods to determine that a satisfactory fit test of your CleanSpace Respirator has been achieved.

1. Warnings & Limitations

CleanSpace is a powered air purifying respirator (PAPR) and should be used with the battery charged and in Power-On mode. CleanSpace Respirators are intended to protect against substances and mixtures which are hazardous to health.

If your CleanSpace Respirator is intrinsically safe, it will be clearly marked as such. Do not use non-intrinsically safe respirators in flammable or explosive atmospheres. Do not use in IDLH, or in oxygen enriched or deficient atmospheres.

Do not submerge the respirator in water. Do not use compressed air or a brush to clean the filter.

Operating Conditions: 0% - 90% RH, -10°C to 45°C. Storage Conditions: 0% - 75% RH, 10°C to 30°C.

Manufacturers Minimum Design Condition/Duration: CleanSpace Respirators will maintain positive pressure in the mask up to 120 l/min. Minimum 4-hour duration. The user must do a Flow Test to check the design condition before each use.

During pre-use visual inspection, if parts are missing or damaged replace them with approved parts before proceeding.


European users should be aware that all markings on the filter refer to the standard EN 12942 (1998) + A2 (2008).

2. System Description


2.1. ALARMS

- A. **Filter Change Alert** Two (2) beeps, Filter Alarm Light will flash. Change filter before entering the contaminated area.
- B. **Low Battery Alarm** Three (3) beeps. Less than 15 minutes runtime. Leave the contaminated area immediately.
- C. **Filter Absent Alarm (CST1002 & CST1012)** – Constant beep, Filter Alarm Light will flash. Install filter before entering the contaminated area.


2.2. BATTERY, FILTERS & PRE-USE PROCESS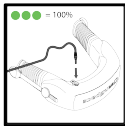


Charge Battery

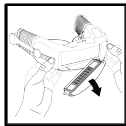Replace Filter

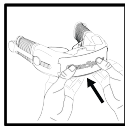Visually Inspect all Parts Before Use

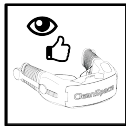Recalibrate Unit

Flow Test: Battery & Filter Lights Flash = FAIL

3. Donning the CleanSpace PAPR

Half Mask	 Assemble	 Turn On	 Don	 Adjust
Full Face Mask	 Assemble	 Turn On	 Don	 Adjust

4. Care and Cleaning

Cleaning or disinfectant wipes can be used to wipe down the components. Use a clean cloth to remove any cleaning residue before use. The mask, harness and neck support can be washed with a mild detergent in warm water. Power units must not be submerged in water. CleanSpace filters cannot be cleaned. Unopened filters have a storage life of five (5) years.

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