

PRIOR TO ENTERING PATIENT'S ROOM

1. PREPARING PAPR	<ol style="list-style-type: none"> 1. Remove Power Unit from Charging & Storage Station. Remove Blue Cleaning & Storage Plug from Power Unit. 2. Check you have the correct mask size, harness and neck support. And a new filter 3. Check battery charge: Press "Power" button. More than 2 Green LEDs should be on. 4. Check Power Unit, mask, harness for damage or missing parts 5. Inspect the new filter. Insert the new filter to the back of the Power Unit and lock in place. 6. Conduct a flow test. Please Flow Test Button. The unit will blow for 15 seconds and more than 2 Green LEDs will be on indicating adequate airflow. If this fails, place Power Unit to the side and prepare another Power Unit. 7. Connect harness to the mask and adjust harness for a comfortable fit. Connect one side of the mask to the Power Unit.
--------------------------	--

2. DON PAPR & PPE	<ol style="list-style-type: none"> 1. Hand hygiene 2. Prepare a hood and gown. Place the power unit into Standby mode (Green LEDs on. No airflow). 3. In Standby mode, place the Power Unit around the neck and mask resting under the chin. 4. Connect the 'open side" so both sides of the mask are connected to the Power Unit. Pull the mask up and take a few breaths. Adjust the Power Unit and mask and pull the harness up to sit comfortably on the crown of your head. 5. Adjust the CleanSpace Respirator for comfort (not too loose or over tight) 6. Hand Hygiene 7. Don hood - ensure the back of the hood remains outside the gown 8. Don gown and gloves
------------------------------	--

EXITING PATIENT'S ROOM

1. IN PATIENT'S ROOM	<ol style="list-style-type: none"> 2. Remove gloves and discard into biohazard bin inside room 3. Hand hygiene 4. Remove gown inside room 5. Hand hygiene 6. Leave patient's room
-----------------------------	--

2. OUTSIDE PATIENT'S ROOM	<ol style="list-style-type: none"> 3. Go to PAPR Cleaning Trolley with Biohazard Bin 4. Put on clean gloves 5. Wipe visor of PAPR with ALCOHOL wipe. Discard wipe 6. Remove hood forward and downward 7. Remove CleanSpace Respirator by removing the harness (forward), mask off (note; Power Unit will continue to blow for 10 seconds), find the connection clip and unclip the mask on one side. Remove the unit/mask from around the neck and place on the trolley. 8. Hand hygiene. 9. Remove the disposable cover on the power unit and discard into the Biohazard Bin. IF REQUIRED, dispose of the filter into the Biohazard Bin by lifting the lock and tipping the filter out over the bin. There is no need to touch the filter. 10. Hand Hygiene
----------------------------------	--

CLEANING & CHARGING

**CLEANING
THE
EQUIPMENT**

**USING
ALCOHOL &
NON-
ALCOHOL
WIPE**

1. Put on new pair of gloves
2. Clean the VISOR with alcohol wipes
3. Use another alcohol wipe to clean HOOD
4. Use another alcohol wipe to clean POWER UNIT
5. Use another alcohol wipe to clean MASK & HARNESS
6. Remove gloves
7. Hand hygiene
8. Use new pair of gloves
9. Clean inside of Mask with NON-alcohol wipe
10. Remove gloves. Hand hygiene
11. Use a new pair of gloves
12. Fit the Cleaning & Storage Plug (Blue) to the Power Unit
13. Place Power Unit into Charging & Storage Station and plug in charging port. Green LEDS will flash to indicate charging
14. Use non-alcohol wipe to clean surface top of trolley

CLEANSPACE TRAINING VIDEOS:

Preparation of the CleanSpace

Donning

Doffing

Cleaning the Equipment

Using the Charging and Storage Station

Replacing the filter

For more information visit website: www.cleanspacehealth.com or email our customer support team: sales@cleanspacetechnology.com