

This Supplement is used to check washing and disinfecting processes for material compatibility with CleanSpace respirator components.

Where biological contaminants are a risk, cleaning and disinfecting a CleanSpace respirator is advisable. CleanSpace recommends deferring to the regulatory authority or standards in your region, to ascertain efficacy of microbiological removal. CleanSpace has not specifically evaluated the effectiveness of any agents with regards to inactivating viruses on the equipment.

Consider rinsing and drying components thoroughly to avoid any residue that may cause irritation to the wearer after use of any non-neutral cleaning or disinfectant agent treatment (including alcohol).

CleanSpace filter media may not be cleaned or disinfected. If a filter change is not required, in a healthcare setting, it is recommended to keep the filter in place. The external hard surface of the filter may be wiped down with a clean damp cloth soaked in a compatible solution or wipe.

Cleaning with solvents is not recommended. Solvents could cause damage to plastic components including cracking, fogging, fading and decreased strength. If equipment is exposed to solvents, rinse well and check parts for cracking or fogging. Please contact the CleanSpace team if you have any questions: sales@cleanspacetechnology.com.

Rinse Aid or Drying Aid is not recommended to be used as an additive to the cleaning cycle. Use of a drying aid may discolour or damage the polycarbonate on the mask valve set.

The prolonged use of any cleaning/disinfecting agent could potentially shorten the useful life of the products. It is imperative that users perform a visual inspection prior to each use to ensure equipment is free from damage. If any visible deterioration is apparent (cracking, crazing, tears etc.), the component should be discarded and replaced. A slight discolouration of silicone components may occur and is acceptable.

Reassemble the mask (including the valve assembly) prior to use according to the User Instructions.

Store CleanSpace respirator components in a dry, dust-free environment away from direct sunlight.

CleanSpace cannot give assurances that deviations from the procedures listed in this supplement and their effects on the performance or safety of the product will be acceptable.











When using detergents, disinfectants, or sterilisation agents, always follow the manufacturer's User Instructions. In the event of conflict, this supplement takes precedence.



Compatible ✓ Not Compatible ✗	EPA Reg. No.	PRO Power Unit	ULTRA Power Unit	All CleanSpace Filters (Note 1.)	Neck Support	Half Mask	Head Harness	Full Face Mask	Full Face Mask Removable Orinasal Piece	Cleaning & Storage Plug	Half Mask Fit Test Adaptor
		CST1001 CST1002	CST1011 CST1012	N/A	CST1019 CST1020	CST1034 CST1035 CST1036	CST1021	CST1017 CST1018	PAF-1022 PAF-1027	CST1024	PAF-0025
COMPATIBLE DETERGENTS											
ASP ENZOL		✓	✓	✓	✓	✓	✓	✓	✓	✓	✓
Borer deconex 41 STS ALKA		✓	✓	✓	✓	✓	✓	✓	✓	✓	✓
Diversey Oxivir Five 16 Concentrate		✓	✓	✓	✓	✓	✓	✓	✓	✓	✓
Ecolab Neutral Disinfectant Cleaner		✓	✓	✓	✓	✓	✓	✓	✓	✓	✓
Getinge Neutrawash		✓	✓	✓	✓	✓	✓	✓	✓	✓	✓
Proctor and Gamble Dawn		✓	✓	✓	✓	✓	✓	✓	✓	✓	✓
STERIS Prolystica® 2X Concentrate Alkaline Detergent		✓	✓	✓	✓	✓	✓	✓	✓	✓	✓
STERIS Prolystica HP Enzymatic Manual Cleaner		✓	✓	✓	✓	✓	✓	✓	✓	✓	✓
STERIS Prolystica® Ultra Concentrate HP Neutral Detergent		✓	✓	✓	✓	✓	✓	✓	✓	✓	✓

Note 1. External Filter Case Only. Filter Media must not be cleaned.



Compatible ✓ Not Compatible ✗	EPA Reg. No.	PRO Power Unit	ULTRA Power Unit	All CleanSpace Filters (Note 1.)	Neck Support	Half Mask	Head Harness	Full Face Mask	Full Face Mask Removable Orinasal Piece	Cleaning & Storage Plug	Half Mask Fit Test Adaptor
		CST1001 CST1002	CST1011 CST1012	N/A	CST1019 CST1020	CST1034 CST1035 CST1036	CST1021	CST1017 CST1018	PAF-1022 PAF-1027	CST1024	PAF-0025
COMPATIBLE CLEANING & DISINFECTING AGENTS											
Ethanol (excluding Rinse-Aid products)		✓	✓	✓	✓	✓	✓	✓	✓	✓	✓
Glutaraldehyde		✓	✓	✓	✓	✓	✓	✓	✓	✓	✓
Hydrogen Peroxide ≤ 100%		✓	✓	✓	✓	✓	✓	✓	✓	✓	✓
Isopropyl Alcohol		✓	✓	✓	✓	✓	✓	✓	✓	✓	✓
Rinse or drying aid		✗	✗	✗	✗	✗	✗	✗	✗	✗	✗
Sodium Hypochlorite (Bleach) ≤ 1% Concentration		✓	✓	✓	✓	✓	✓	✓	✓	✓	✓
Sodium Hypochlorite (Bleach) 1 - 10% Concentration		✗	✗	✓	✓	✓	✓	✗	✓	✓	✓
Quaternary Ammonium ²		✓	✓	✓	✓	✓	✓	✓	✓	✓	✓

Note 1. External Filter Case Only. Filter Media must not be cleaned.

Note 2. Rinse after applying to avoid skin irritation.



Compatible	Not Compatible	EPA Reg. No.	PRO Power Unit	ULTRA Power Unit	All CleanSpace Filters (Note 1.)	Neck Support	Half Mask	Head Harness	Full Face Mask	Full Face Mask Removable Orinasal Piece	Cleaning & Storage Plug	Half Mask Fit Test Adaptor
			CST1001 CST1002	CST1011 CST1012	N/A	CST1019 CST1020	CST1034 CST1035 CST1036	CST1021	CST1017 CST1018	PAF-1022 PAF-1027	CST1024	PAF-0025
COMPATIBLE BRANDS												
	Actisan-5 L											
	APESIN AP3								On request			
	AzoMax Wipes		On request	On request	On request	On request	On request	On request	On request	On request	On request	On request
	CaviWipes™	46781-8							On request			
	CaviWipes™ Bleach - disinfecting towelettes	46781-14							On request			
	CaviWipes1™	46781-13							On request			
	CleanCide Wipes	34810-36							On request			
	clinell® UNIVERSAL WIPES								On request			
	Clorox® Disinfecting Wipes	67619-31							On request			
	Clorox Healthcare® Bleach Germicidal Wipes	67619-12							On request			
	Clorox Healthcare® Hydrogen Peroxide Cleaner Disinfectants	67619-25							On request			
	Dismozon®											

Note 1. External Filter Case Only. Filter Media must not be cleaned.



Compatible ✓ Not Compatible ✗	EPA Reg. No.	PRO Power Unit	ULTRA Power Unit	All CleanSpace Filters (Note 1.)	Neck Support	Half Mask	Head Harness	Full Face Mask	Full Face Mask Removable Orinasal Piece	Cleaning & Storage Plug	Half Mask Fit Test Adaptor
		CST1001 CST1002	CST1011 CST1012	N/A	CST1019 CST1020	CST1034 CST1035 CST1036	CST1021	CST1017 CST1018	PAF-1022 PAF-1027	CST1024	PAF-0025
COMPATIBLE BRANDS (Continued)											
Dispatch® Hospital Cleaner Disinfectant Towels with Bleach	56392-8	✓	✓	✓	✓	✓	✓	On request	✓	✓	✓
Ecolab Klerwipe™ Sporicidal Low Residue (Peroxide)		✓	✓	✓	✓	✓	✓	✓	✓	✓	✓
Ecolab Klercide™ Sporicidal Active Chlorine	1677-254	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓
Ecolab Klercide™ 70/30 (IPA) (Isopropyl Alcohol)	1677-249	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓
Ecolab Incidin™ Plus		✓	✓	✓	✓	✓	✓	✓	✓	✓	✓
germex™B-12		✓	✓	✓	✓	✓	✓	✓	✓	✓	✓
Non-alcoholic cleaning wipes (eg. benzalkonium chloride)		✓	✓	✓	✓	✓	✓	On request	✓	✓	✓
OPTIM 1	74559-10-83259	✓	✓	✓	✓	✓	✓	On request	✓	✓	✓
Oxivir® Tb Wipes	70627-56	✓	✓	✓	✓	✓	✓	On request	✓	✓	✓
Sani-Cloth® AF3 Germicidal Disposable Wipes	9480-9	✗	✗	✓	✓	✓	✓	On request	✓	✓	✓
Super Sani-Cloth® Germicidal Disposable Wipes	9480-4	✗	✗	✓	✓	✓	✓	On request	✓	✓	✓
Sani-Cloth® Bleach Germicidal Disposable Wipe	9480-8	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓
Sani-HyPerCide™ Germicidal Disposable Wipe	9480-16	✓	✓	✓	✓	✓	✓	On request	✓	✓	✓

Note 1. External Filter Case Only. Filter Media must not be cleaned.



Compatible ✓ Not Compatible ✗	EPA Reg. No.	PRO Power Unit	ULTRA Power Unit	All CleanSpace Filters (Note 1.)	Neck Support	Half Mask	Head Harness	Full Face Mask	Full Face Mask Removable Orinasal Piece	Cleaning & Storage Plug	Half Mask Fit Test Adaptor
		CST1001 CST1002	CST1011 CST1012	N/A	CST1019 CST1020	CST1034 CST1035 CST1036	CST1021	CST1017 CST1018	PAF-1022 PAF-1027	CST1024	PAF-0025
COMPATIBLE BRANDS (Continued)											
Tristel Duo Wipes		✓	✓	✓	✓	✓	✓	On request	✓	✓	✓
Tristel Trio 50 Wipes		✓	✓	✓	✓	✓	✓	On request	✓	✓	✓
Tuffie 5 Universal Wipes		✓	✓	✓	✓	✓	✓	✓	✓	✓	✓
V-Wipes (Whitely Medical)		✓	✓	✓	✓	✓	✓	On request	✓	✓	On request

Note 1. External Filter Case Only. Filter Media must not be cleaned.

