




## CleanSpace<sup>®</sup> SMART App | Preparation for Data Insights Reports

### DOWNLOAD SMART APP




### CHECK SMART APP VERSION

#### iOS

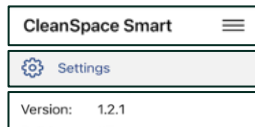

- Click three lines on top right of the Main page
- Select 'Settings'
- About App > Confirm **Version: 1.2.1**

#### Android

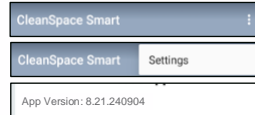

- Click three dots on top right of the Main page
- Select Settings
- Confirm **App Version 8.21.240904**

### PAIR RESPIRATOR

*Important: Confirm Bluetooth is turned ON in mobile device settings.*

- On main page select PAIR UNIT (Unit Pairing for iOS) and select the respirator that requires pairing.
- Make sure that the respirator is in standby mode by pressing the power button once.
- Respirators available for device pairing are listed in order of signal strength.
- When paired the respirator will beep, the SET UP & CHECK page will automatically display.



### CHECK RESPIRATOR FIRMWARE

#### iOS & Android


- Go to DASHBOARD to check Firmware
- Firmware : **REV4.1.0**
- Update if required (Refer to CleanSpace Updating Firmware Supplement)



### SYNCHRONISE DATA


*Important: Make sure the mobile device is connected to the internet to allow for data transfer.*

#### iOS

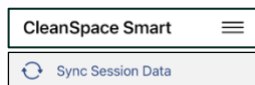

- Click three lines on top right of the Main page
- Select 'Synch Session Data'

#### Android

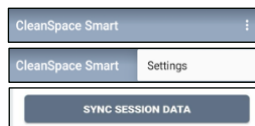

- Click three dots on top right of the Main page
- Select Settings
- Select 'Synch Session Data'

### INFORMATION REQUIRED FOR DATA INSIGHTS REPORT

Please provide this information to your CleanSpace contact.

- Company Name
- Contact Person
- Respirator Serial Number(s) ONLY (no user names)
- Date of Respirator purchases
- Date of Data synchronisation
- Mobile device type used for synchronization (iOS or ANDROID)

