


CleanSpace Respirators - Quick Troubleshooting Flowchart

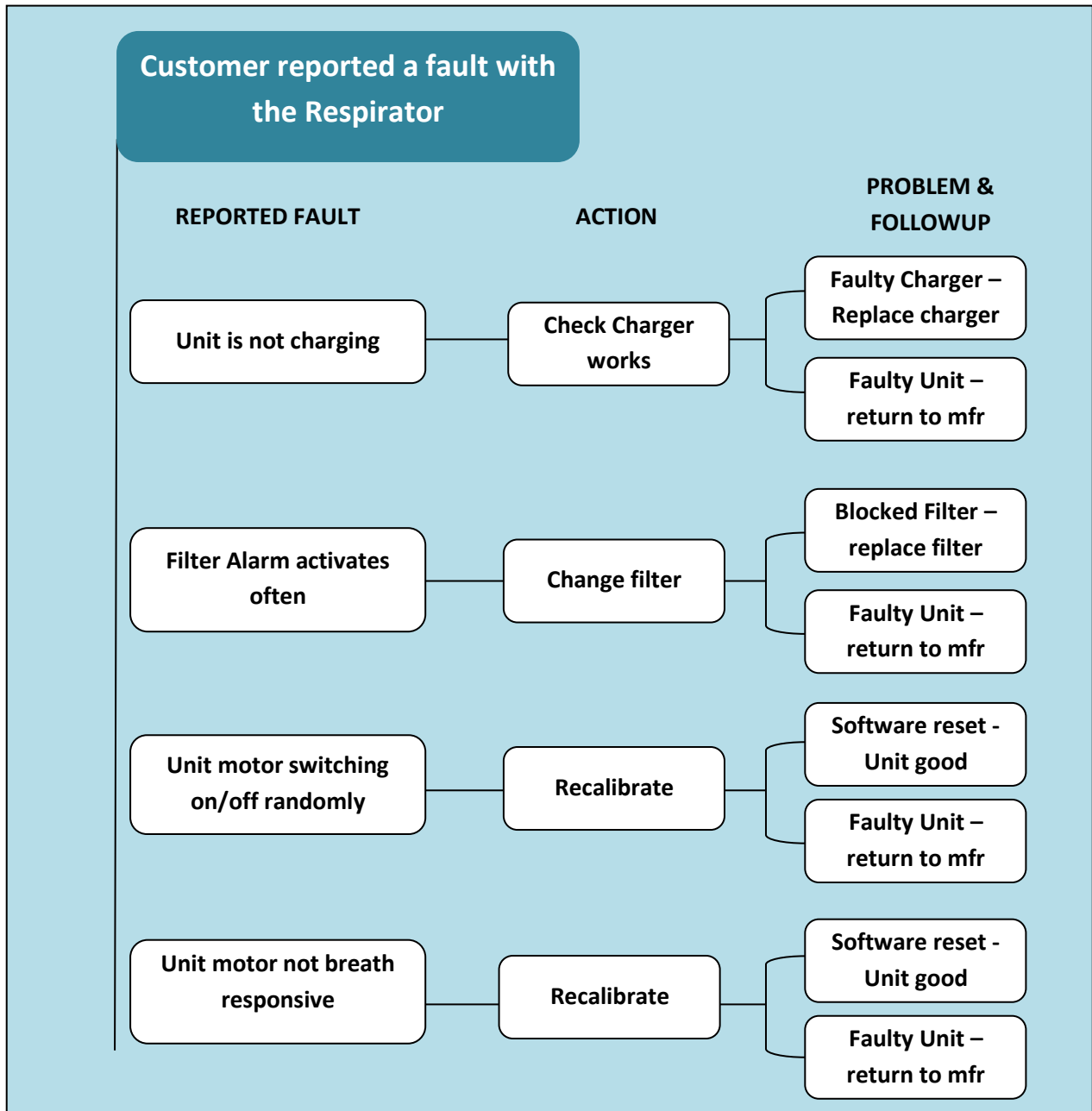

STEPS FOR RECALIBRATING THE RESPIRATOR

To perform a recalibration on your CLenSpace2® respirator follow these steps:

1. Place a finger over the Power Button and Airflow Button at the same time.
2. Press and hold the Buttons at the SAME TIME.
3. The buttons should be held until both the Blue and Red lights come on (approximately 1-3 seconds), after which the Buttons should be released

The lights will stay on for 10 seconds while the respirator recalibrates itself. The recalibration is completed when the blue and red lights switch off.